


Sileo Aged Care Bed – Back Up Battery

INSTALLATION

A rechargeable battery designed for a single operation in the event of a power failure.

- ⚠ Please read these instructions before installation
- ⚠ Disconnect from mains power before install or uninstall Back Up Battery
- ⚠ Back Up Battery cannot be serviced
- ⚠ The battery must be charged for at least 10 hours every 4-5 months. Battery will recharge when bed is connected to a mains power source.

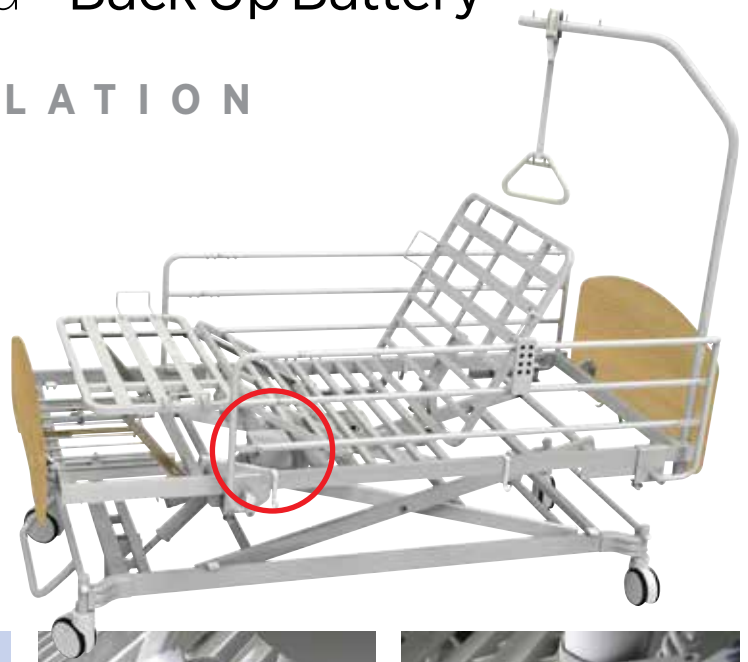

No tools are required to fit the Back Up Battery



1 To access the cradle that holds the Back Up Battery, use the hand controller to raise the leg rest, then fully extend the heel platform by manually lifting upwards.



2 Locate the Back Up Battery cradle, positioned underneath the foot rest panel.



3 Slide the Back Up Battery into the bracket and push until an audible 'click' is heard. The battery only fits into the cradle one way.



4 To connect the Back Up Battery to the Control Box, plug the power cord into the appropriate slot (indicated with the black battery icon:).



5 To avoid a trip hazard or damage to the cord, wrap and tie any excess cord with a cable tie.



To remove battery (for replacement or cleaning) unplug the Back Up Battery from the control box. Press the two clips on the side of the Battery (through the bracket) and pull up the battery upwards.

Storage

Store in a cool, dry location out of direct sunlight

Warranty


Novis Healthcare Pty Ltd warrants this product for a period of 12 months.



Please refer to the Sileo Aged Care Bed Warranty Statement for full details.

Sileo Aged Care Bed – Head and Foot Panels

INSTALLATION

Replacement head and foot panels.

 Please read these instructions before installation

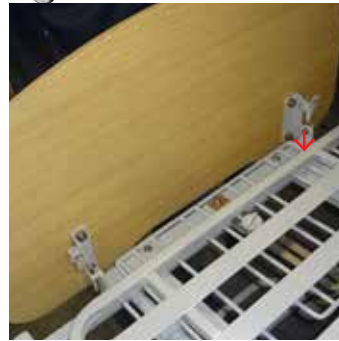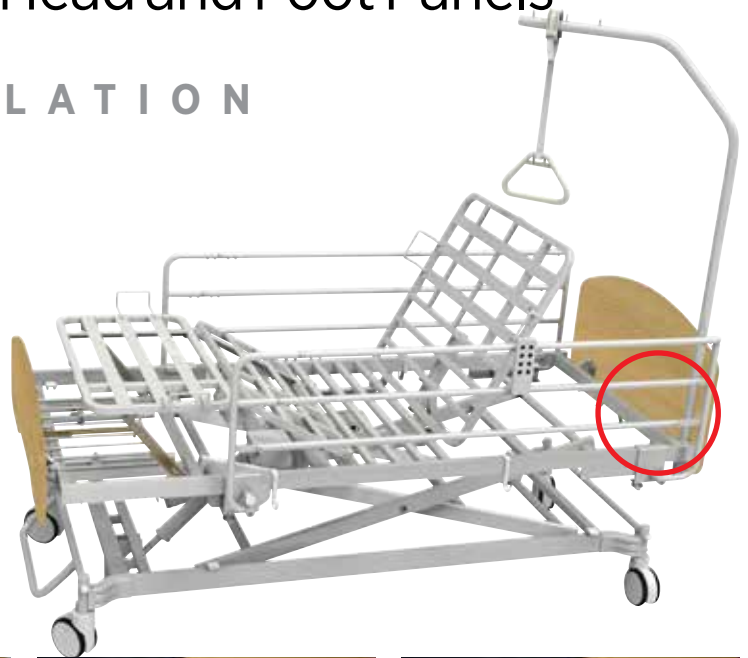

No tools are required to fit or remove Head and Foot Panels.

1 Release the two locks on the inside of the panel by lifting the lever (as indicated by the green arrow)

2 With locks facing inwards, hook the top clip on each side of the panel into the corresponding holes on the bed frame.

3 Once secure, push down hard on each lock until an audible click is heard, and the locking mechanism is secure.

Removal

To remove head and foot panels, release the locks and remove each panel carefully.

Storage

Store in a warm, dry location out of direct sunlight.

Cleaning

Clean as per instructions included with the Sileo Aged Care Bed User Guide (page 14).

Warranty

Novis Healthcare Pty Ltd warrants this product for a period of 2 years.


Please refer to the Sileo Aged Care Bed Warranty Statement for full details.

Sileo Aged Care Bed – Self Help Pole

INSTALLATION

Support pole to aid patient repositioning while lying in bed. Connect the handle to the pole and insert into the round hole at either side of the head end of the bed.

Please read these instructions before installation and use

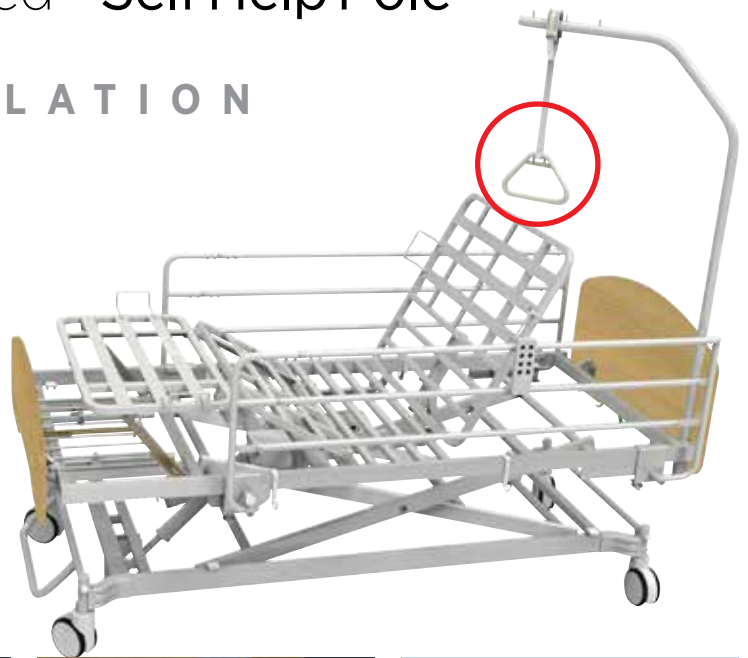

No tools are required.



1 Insert the base of the pole into the inbuilt bracket located at the head-end of the bed frame.



The Self Help Pole has a notch to prevent the pole from swinging sideways during transfer. To rotate the position of the pole, lift slightly, turn and reinsert.



The height of the handle can be adjusted to suit the individual use. For ideal height, the patient should be able to reach the handle from a reclined position.

Safety

- Do not exceed the Safe Working Load (SWL) of 75kg
- Ensure the pole is swung out of the way when not in use or when seated on the side of the bed
- Use caution with existing upper limb pain or injuries
- Do not hang other items off the pole

Storage

Store in a cool, dry location out of direct sunlight.

Cleaning

Refer to the Sileo Aged Care Bed User Guide ([page 14](#)).

Warranty

Novis Healthcare Pty Ltd warrants this product for a period of 2 years.

Please refer to the Sileo Aged Care Bed Warranty Statement for full details.


To shorten or raise the handle

Pull down on the strap 'tail'



To lengthen or lower the handle

Lift the metal buckle until horizontal, then push the 'tail' through the buckle towards the handle.

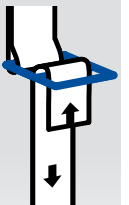


Sileo Aged Care Bed – Collapsible Side Rails

INSTALLATION

Collapsible metal side rail system attaches securely to side of the bed frame.

- Please read these instructions before installation and use
- Minimum height from top of mattress to top of rail must be 220 mm.

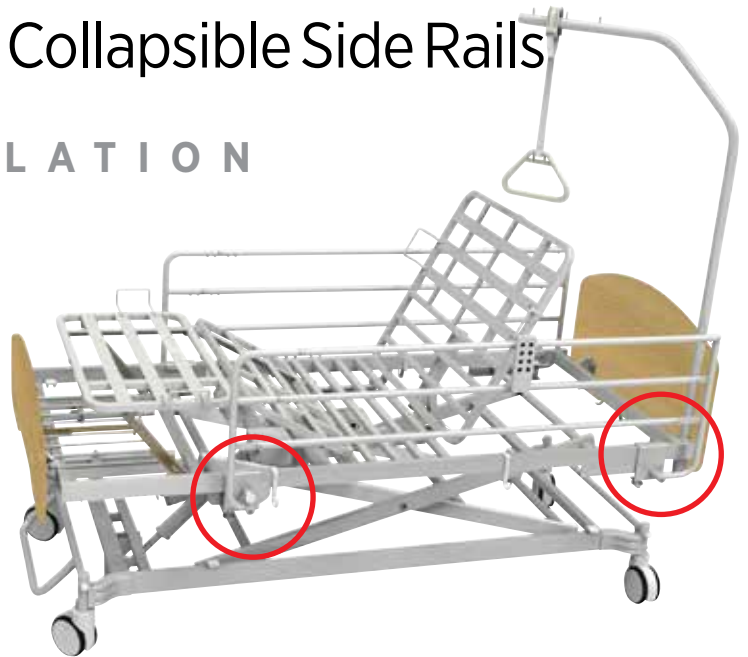

- 1 With the side rails upright (the un-collapsed position), align the rail alongside the bed frame in the correct orientation – the smaller bracket with the securing pin at the foot end; the larger bracket to the head end.
- 2 Connect the side rail onto the bed by securing the brackets over the side of the frame.
Note, bracket hook and second star knob will be positioned downwards.
- 3 At the foot end, align the bracket hole with the nut hole welded to the frame. Insert the star knob and turn clockwise to tighten.
- 4 Locate the second star knob positioned underneath the bracket, and turn clockwise to tighten.



- 5 Repeat this process at the head end, securing both star knobs until firm.

To lower the rail, release the locking pin located at the foot end of the rail by pulling outwards. Once disengaged, gently push the rail downwards towards the foot of the bed.

To raise the rail, lift the rail upwards until the lock pin engages (an audible click will be heard). Always ensure the rail lock is secure.

Safety

Do not sit on or place heavy loads on the side rails.

Cleaning

Refer to the Sileo Aged Care Bed User Guide ([page 14](#)).

Storage

Store in a warm, dry location out of direct sunlight

Warranty

Novis Healthcare Pty Ltd warrants this product for a period of 2 years.

Please refer to the Sileo Aged Care Bed Warranty Statement for full details.